PuroTM suoja

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English

What Puro™ Suoja is and what it is intended for

Puro™ Suoja is a lipid microemulsion eye drop intended for treating and preventing symptoms and signs of moderate and severe dry eye. Small spots of very high osmolarity in the tear film of dry eyes induce damage and inflammation of the ocular surface. It is the damaged ocular surface, left without proper protection by the tear film that causes uncomfortable sensations of itching, burning and grittiness of the eyes, and blurred vision. The Puro™ Suoja microemulsion has been designed to counteract all major factors that maintain dry eye. Once the stability and osmolarity of the tear film have first been normalised, the ocular surface starts recovering from damage. Hyaluronan, trehalose, and glycerol bind water, stabilise, and moisturise the tear film, and with the mildly hypotonic (low-osmolar) solution they help keep osmolarity at a safe level. Sacha inchi seed oil strengthens the tear film lipid layer to prevent evaporation and hyperosmolarity and lowers the surface tension to restore its rapid spreading. The oil and the high-molecular-weight hyaluronan improve the retention of the protective tear film intact on the ocular surface. Hyaluronan, trehalose, and natural antioxidants of sachu inchi seed oil also assist ocular cells in recovering from inflammatory damage. Settled inflammation decreases redness in the ocular conjunctiva. In most patients with severe dry eye, the eyes should feel significantly more comfortable after 10 to 30 days; this effect, proved in a clinical trial, can be achieved with frequent use of the eye drops as instructed below. Also, vision will be improved in eye-straining conditions. Manufactured by the proprietary DEMET® microemulsion technology developed by Oy Finnsusp Ab, the solution is optically clear and with optimal viscosity (no shaking of the bottle is needed and no blurring of vision), with optimal surface tension, potassium ion concentration, and omega-6/omega-3 unsaturated fatty acid ratio to support the healthy condition of your eyes. Naturally, it is free from preservatives and can be used with contact lenses – the lenses need not be removed from the eyes. The bottle does not contain materials releasing silver ions into the eye. Puro™ Suoja is a sterile CE marked medical device.

Instructions for use

Tear off and discard the seal of the cap to open the bottle for the first time. Open the cap. Practise handling of the bottle before the first application. Gently squeeze the bottle in tip down position only. Squeeze the bottle with increasing force until a

drop builds up at the tip and falls off. Apply 1 drop in both eyes 3–4 times daily. Shake the remaining drop away from the tip and then close the cap. Do not put the cap on until the bottle has got its original shape (replacement air is flowing in the bottle). A small liner pad inside the cap ensures that the next drop you take will be fresh and clean. The contents should be used within six (6) months from opening and by the expiry date.

Precautions and warnings

- Do not use, if the bottle is damaged.
- The bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with fingers.
- Do not squeeze the bottle with cap on.
- Do not use after expiry date printed on the bottle and carton box or after discard date (6 months after opening).
- If you experience blurring of vision just after application, please wait a moment before continuing activities that require clear vision. Blurring should be mild and temporary only.
- Wait at least 15 minutes between possible applications of other ophthalmic medication.
- After application, do not put the cap on until the bottle has got its original shape (replacement air is flowing in the bottle).
- Keep out of the reach and sight of children.
- Store at 5–30 °C. Ensure the recommended storage conditions at all times. Protect from direct sunshine and freezing. Store at recommended conditions at all times.
- Discard the empty bottle appropriately.
- The manufacturer, Oy Finnsusp Ab, quality@piiloset.fi / tel. 02 489110 and, Finnish Medicines Agency Fimea, tel. 029 522 3341, must be notified of serious adverse events related to use of the product.

Contraindications

Do not use if you are sensitive to any of the ingredients. If in doubt, consult a healthcare professional.

Composition

0.1% sachu inchi (*Plukenetia volubilis*) seed oil, 2% trehalose, 0.2% high-molecular-weight hyaluronan, glycerol and emulsifiers polyoxyethylene sorbitan monooleate and sorbitan monooleate in sterile and hypotonic trometamol/citrate buffer emulsion. Contains no preservatives, alcohol, phosphates, borates, mineral oil, or materials of animal origin.

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Recycle packing materials

Sort box into carton waste container, leaflet into paper waste container and dispenser into plastic packaging waste container or energy waste.