TM
Puro
tyrni

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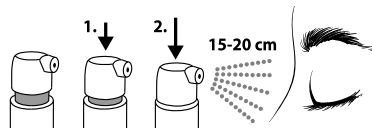
English

What Puro™ Tyrni is and what it is intended for

Puro™ Tyrni is a lipid microemulsion eye spray intended for treating and preventing symptoms of dry eyes as well as for moisturizing the eyelids and the skin around the eyes. Always apply the spray on closed eyelids. The microemulsion contains sea buckthorn seed oil that spreads on the ocular surface, strengthening the protection of the natural tear film lipid layer and preventing evaporation of the tear fluid. Lowering the surface tension of the tear film improves its retention and spreading on the ocular surface. Puro™ Tyrni relieves and prevents irritation, dryness, burning and watering of the eyes caused by various environmental factors such as dry air, wind, smoke, air pollution, intensive light, as well as by ocular surgery, allergies, or contact lens wear. Hyaluronan, glycerol and natural antioxidants of sea buckthorn seed oil protect the ocular surface and the skin around the eyes from cell injuries caused by drying and speed up their healing. Hyaluronan and glycerol also bind water and moisturize the surface of the eyes. The microemulsion eye spray refreshes eyes that are strained due to mobile device use, visual display work, reading, or driving. Puro™ Tyrni is a sterile CE marked medical device developed, manufactured, and clinically tested in Finland. It does not contain preservatives or alcohol and can be used with contact lenses and eye make-up. Thanks to its special design, the dispenser can be used in any position.

Instructions for use

Take off the protection cap. Before the first application, test the spraying mechanism a few times on your arm or in the air. Notice that there is a small step in the nozzle; to apply the spray, press the nozzle fully down. **Next, hold the bottle approximately 15–20 cm from the eyes. Close both eyes and then apply 1–2 sprays on the eyelids.**

Put the protection cap back on the nozzle head. You may use the spray several times a day as your symptoms require and during contact lens wear. If you are using other personal care products in the eye area or eye make-up,

let them settle for at least 10 minutes before applying the spray, or use the other product not earlier than 10 minutes after applying the spray. If you are using eye make-up, you may wish to apply the spray from a slightly longer distance to avoid spreading (depending on the type of make-up). Use within six (6) months from opening the bottle and by the expiry date.

Precautions and warnings

- Do not use, if the bottle is damaged.
- Do not touch the eye or contact lens with the nozzle. Do not touch the tip of the nozzle with fingers.
- Do not use after expiry date printed on the bottle and carton box or after discard date (6 months after opening).
- You may experience temporary discomfort, if the spray is applied on an open eye.
- Leave at least 10 minutes between applications of the spray and another ophthalmic medication.
- Keep out of reach and sight of children.
- Store at 5–25 °C, protect from direct sunshine. Do not use the product in the event it has frozen or it has been left in a hot place for a long time.
- Discard the empty bottle appropriately. Due to intentional overfill, a small volume of solution will remain in the bottle.
- The manufacturer, Oy Finnsusp Ab, quality@piiloset.fi / tel. 02 489110 and, Finnish Medicines Agency Fimea, tel. 029 522 3341, must be notified of serious adverse events related to use of the product.

Contraindications

Do not use if you are sensitive to any of the ingredients. If in doubt, consult a healthcare professional.

Composition

0.4% Sea buckthorn (*Hippophae rhamnoides*) seed oil, hyaluronan, glycerol and emulsifiers polyoxyethylene sorbitan monooleate and sorbitan monooleate in sterile hypotonic trometamol/citrate buffer emulsion. Contains no preservatives, alcohol, phosphates, borates, mineral oil or materials of animal origin.

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Recycle packing materials

Sort box into carton waste container, leaflet into paper waste container and dispenser into plastic packaging waste container or energy waste.